

# Dandy Digital Fast Track

Learn the skills and techniques that lead to outstanding patient outcomes.

## DANDY DIGITAL FAST TRACK

START ..... WEEK 1 ..... WEEK 2 ..... WEEK 3 ..... WEEK 4 ..

PHASE 1: DIGITAL FUNDAMENTALS		PHASE 2: LIVE CASES WITH LIVE PATIENTS		
Equipment set-up & Wellness Scans 	Live Virtual Launch Training 	Live First Case Monitoring 	Simple Cases 	Expand Your Horizons 
<b>WEEKLY SUPPORT &amp; COACHING</b>				

## PHASE 1: Digital Fundamentals

WEEK 1

### Week 1: Pre-work checklist

MUST BE DONE BEFORE LAUNCH TRAINING

When your equipment arrives, you will receive detailed instructions on how to set up your equipment and start learning the digital workflow.

**The steps below must be completed before your scheduled launch training!**

#### 1 Set up

Get your accounts, laptop, and scanner set-up and ready for action.

- Set up Doctor Profile in Dandy Portal | ⌚ 5 MINUTES
- Set up laptop and scanner | ⌚ 15 MINUTES
- Calibrate and sterilize the scanner | ⌚ 5 MINUTES

#### 2 Learn

Level up your scanning technique and learn the digital workflow.

- Learn the scanning workflow | ⌚ 15 MINUTES

#### 3 Scan

Practice Dandy's scanning workflow on the provided model.

- Wellness Scans | ⌚ 30 MINUTES

Scan the QR code to access tutorials for pre-work steps:

- ✓ 1 Set up
- ✓ 2 Learn
- ✓ 3 Scan



## Virtual launch training

🕒 2 HOURS

In this hands-on, personalized session, your trainer will guide you and your staff through Dandy's world-class digital workflow. You will learn the techniques and step-by-step process that leads to precise-fitting restorations. Staff members who attend live are eligible to receive 2 CE credits.

- ✓ Dandy's digital Crown & bridge workflow
- ✓ Scan techniques and best practices
- ✓ 1-on-1 scan coaching for each individual
- ✓ Dandy Support introduction

### PHASE 2: Live Cases with Real Patients

WEEKS 2-4

Over the next 3 weeks, your Dandy Launch Trainer will be virtually monitoring your progress, delivering personalized feedback, and helping you deliver successful patient outcomes!

## Week 2: Live first case monitoring

During week 2, you will begin treating patients using the Dandy digital workflow! A Dandy expert will be with you virtually for your first case. We will watch and review your scans, provide feedback on your preparation, and answer any questions in our live chat window.

## Week 3: Simple cases

During week 3, we recommend you treat 3 or more patients using Dandy's digital workflow. Make sure to stay with simple cases like single unit posteriors. This week you should try and work in some night guard treatments as well.

## Week 4: Expand your horizons

During week 4, you should be feeling more comfortable with the Dandy digital workflow. We recommend expanding into new treatments such as night guards, bleaching trays, retainers, or clear aligners. These products will help grow your confidence (and grow your practice!)

## Graduation

After you've completed your Dandy Digital Fast Track, you will become a Dandy Certified Digital Dentist! Be proud of what you've accomplished, you have now successfully learned the workflow and techniques to effectively treat your patients with Dandy's world-class digital workflow. After graduation, consider booking an advanced product training to learn digital workflows for Implants, Partial Dentures or Full Dentures with Dandy!