INSTRUCTIONS FOR USE (IFU)

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MANUFACTURER

Manufacturer	Zima International, Inc. Also DBA Dandy
Address	1320 N. 300 W., Lehi, UT 84043, United States
Contact / Website	www.meetdandy.com
Complaints	complaints@meetdandy.com

DEVICE IDENTITY

Device Identity

Device Family	Orthodontic Retainer
GMDN / EMDN Code	35310
Product Code(s)	RTRNLNGF, RTRNZEND, RTRNHLWY, THERMOER
Device Type	Custom-made dental device (CMD) / Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reuse	Reusable
Intended User	Prescriber: Dental professional / dental practice

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AUSTRALIA (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

Australia — Patient Matched Medical Device (PMMD)

GMDN / EMDN	35310
Product Code(s)	RTRNLNGF, RTRNZEND, RTRNHLWY, THERMOER
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
Australian Sponsor	Name: DANDY Labs AUS Pty Ltd Company No.: ABN: 33693955761 Address: Tower One - International Towers, Level 46, 100 Barangaroo Avenue, Sydney, New South Wales 2000, Australia
Intended User	Prescriber — Dental professional / dental practice
Device Type	Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

An orthodontic retainer is a dental device designed to maintain the position of teeth following orthodontic treatment. It is used to prevent teeth from shifting and ensure orthodontic treatment results are maintained.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: The retainer is a custom-made dental device that fits over(or behind) the teeth. It is designed to be comfortable and easy to wear.
- Material: Zendura A, Phase Ortho, Stainless Steel

[Retainer Material Safety Data Sheets](#)[Fixed Wire Material Safety Data Sheet](#)**3) Contraindications**

The retainer should not be used if the user has a known allergy to plastic/metal or if they have an existing dental condition that makes it difficult to wear the device

4) Warnings / Precautions / Potential Risks

WARNING

The use of retainers may have potential risks, including breakage. They can be prone to breakage or cracking, especially if they are not handled or cared for properly. Chewing on the retainer or exposing it to excessive heat can increase the risk of damage.

CAUTION

NA

5) Cleaning and Care

Directions for Use

- To ensure the proper use of removable retainers, dental professionals and patients should follow these steps:
- Wash your hands thoroughly before handling the device.
- Carefully insert the device into the patient's teeth, starting from the back of the patient's mouth and working forward.
- Use your fingers to gently press the device onto the patient's teeth to ensure a snug fit.

Cleaning instructions

1. To clean the removable retainer, patients should follow these steps:
2. Use a soft-bristled toothbrush and non-abrasive toothpaste to brush the device gently.
3. Rinse the device thoroughly with cool water.
4. Do not use hot water, as it can warp the device.
5. Do not use mouthwash, as it can discolor the device.
6. Clean the device daily..

Recommended Wear Time

- Wear times for removable retainers can differ based on the clinician and clinical experience.
- Dandy recommends:
 - First 3-6 Months:
 - Retainers should be worn for 22 hours a day or more. The only time they should be removed is to brush and floss or to eat.
 - First 2 Years:
 - The patient will be able to start wearing their retainers only during sleep.
 - Third Year to the Rest of the Patient's Life:
 - Retainers do not have to be worn as frequently; the goal is to attempt to wear them about every other night.

6) Storage

When not in use, store the removable retainer in the provided case. Keep the case in a cool, dry place, away from direct sunlight.

7) Expected Life / Service

Orthodontic retainer life expectancy varies by type, with clear plastic ones lasting 1-3 years, traditional Hawley (wire/acrylic) retainers lasting 5-10 years, and fixed (bonded wire) retainers potentially lasting 10+ years or even decades with excellent care, though all need replacement due to wear, damage, or dental changes; proper cleaning, storage, and avoiding heat/abrasive foods are crucial for longevity.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

Document State: Effective (Simple)

Confidential

Page 5 of 26
FRM-0002002, revision 2

- The Manufacturer and/or Dandy Australia entity using the contacts above, and
- Therapeutic Goods Administration (TGA)

In Australia, serious incidents involving medical devices must be reported to the TGA in accordance with the Therapeutic Goods Act 1989.

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

Orthodontic retainers, especially clear plastic ones, are tough to recycle in regular bins due to multi-layer plastics and medical classification, so the best disposal is through dentist programs for specialized recycling, or as medical/sealed waste if necessary, preventing landfill/ocean pollution.

EUROPEAN UNION (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

European Union — Custom-Made Device (CMD)

GMDN / EMDN	Q010499 - Orthodontic devices - other
Product Code(s)	RTRNLNGF, RTRNZEND, RTRNHLWY, THERMOER
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
EU authorized representative (EURP)	Name: Dandy Labs Europe Company No.: Registration No.: 989857065 Address: 3, boulevard de Sebastopol, 75001 Paris, France
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

An orthodontic retainer is a dental device designed to maintain the position of teeth following orthodontic treatment. It is used to prevent teeth from shifting and ensure orthodontic treatment results are maintained.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: The retainer is a custom-made dental device that fits over(or behind) the teeth. It is designed to be comfortable and easy to wear.
- Material: Zendura A, Phase Ortho, Stainless Steel

[Retainer Material Safety Data Sheets](#)

[Fixed Wire Material Safety Data Sheet](#)

3) Contraindications

The retainer should not be used if the user has a known allergy to plastic/metal or if they have an existing dental condition that makes it difficult to wear the device

4) Warnings / Precautions / Potential Risks

WARNING

The use of retainers may have potential risks, including breakage. They can be prone to breakage or cracking, especially if they are not handled or cared for properly. Chewing on the retainer or exposing it to excessive heat can increase the risk of damage.

CAUTION

NA

5) Cleaning and Care

Directions for Use

- To ensure the proper use of removable retainers, dental professionals and patients should follow these steps:
- Wash your hands thoroughly before handling the device.
- Carefully insert the device into the patient's teeth, starting from the back of the patient's mouth and working forward.
- Use your fingers to gently press the device onto the patient's teeth to ensure a snug fit.

Cleaning instructions

1. To clean the removable retainer, patients should follow these steps:
2. Use a soft-bristled toothbrush and non-abrasive toothpaste to brush the device gently.
3. Rinse the device thoroughly with cool water.
4. Do not use hot water, as it can warp the device.
5. Do not use mouthwash, as it can discolor the device.
6. Clean the device daily..

Recommended Wear Time

- Wear times for removable retainers can differ based on the clinician and clinical experience.
- Dandy recommends:
 - First 3-6 Months:
 - Retainers should be worn for 22 hours a day or more. The only time they should be removed is to brush and floss or to eat.
 - First 2 Years:
 - The patient will be able to start wearing their retainers only during sleep.
 - Third Year to the Rest of the Patient's Life:
 - Retainers do not have to be worn as frequently; the goal is to attempt to wear them about every other night.

6) Storage

When not in use, store the removable retainer in the provided case. Keep the case in a cool, dry place, away from direct sunlight.

7) Expected Life / Service

Orthodontic retainer life expectancy varies by type, with clear plastic ones lasting 1-3 years, traditional Hawley (wire/acrylic) retainers lasting 5-10 years, and fixed (bonded wire) retainers potentially lasting 10+ years or even decades with excellent care, though all need replacement due to wear, damage, or dental changes; proper cleaning, storage, and avoiding heat/abrasive foods are crucial for longevity.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy EU entity using the contacts above, and corresponding local requirements
- The Competent Authority of your Member State, in accordance with Regulation (EU) 2017/745 (MDR).

Competent Authorities:

- France: Agence Nationale de Sécurité du Médicament et des Produits de Santé (ANSM)
- Spain : Spanish Agency for Medicines and Health Products (AEMPS)
- Germany : Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM) (BfArM)
- Italy : Ministero della Salute / Istituto Superiore di Sanità (ISS)
- Spain: Agencia Española de Medicamentos y Productos Sanitarios (AEMPS) for Spain
- Netherlands: Inspectie Gezondheidszorg en Jeugd (IGJ)
- Austria: Bundesamt für Sicherheit im Gesundheitswesen (BASG)
- Sweden: Läkemedelsverket
- Denmark: Lægemiddelstyrelsen
- Portugal: Autoridade Nacional do Medicamento e Produtos de Saúde (INFARMED)

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

Orthodontic retainers, especially clear plastic ones, are tough to recycle in regular bins due to multi-layer plastics and medical classification, so the best disposal is through dentist programs for specialized recycling, or as medical/sealed waste if necessary, preventing landfill/ocean pollution.

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UNITED KINGDOM (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

United Kingdom — Custom-Made Device (CMD)

GMDN / EMDN	35310
Product Code(s)	RTRNLNGF, RTRNZEND, RTRNHLWY, THERMOER
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
UK Responsible Person (UKRP)	Name: DANDY LABS GB, LTD Company No.: Company Number: 16873608 Address: 5 New Street Square, London EC4A 3TW, United Kingdom
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Reusable

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- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: The retainer is a custom-made dental device that fits over (or behind) the teeth. It is designed to be comfortable and easy to wear.
- Material: Zendura A, Phase Ortho, Stainless Steel

[Retainer Material Safety Data Sheets](#)[Fixed Wire Material Safety Data Sheet](#)**3) Contraindications**

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- The Manufacturer and/or UK Responsible Person (UKRP) using the contacts above, and
- The UK Medicines and Healthcare products Regulatory Agency (MHRA) — www.gov.uk/mhra

In accordance with The Medical Devices Regulations 2002 (as amended).

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The table below lists the local Authorized Representative or Sponsor for each country / jurisdiction where this device is marketed. For incident reporting, contact the representative for your country.

La tabla siguiente enumera el Representante Autorizado o Patrocinador local para cada país. | Le tableau ci-dessous liste le Représentant Autorisé local pour chaque pays. | Die folgende Tabelle enthält die bevollmächtigten Vertreter je Land.

Jurisdiction	Authorised Rep / Sponsor	Registered Address / Contact Email	Reg. Number	Regulatory Body
Australia (AUS)	DANDY Labs AUS Pty Ltd	Tower One, Level 46, 100 Barangaroo Ave, Sydney NSW 2000 Compliance@meetdandy.com	ABN: 33693955761	TGA (Therapeutic Goods Administration)
EU — English	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	Competent Authority / EUDAMED
EU — España	Dandy Labs Europa	3, boulevard de Sebastopol, 75001 París, Francia Compliance@meetdandy.com	Reg: 989857065	AEMPS / Autoridad Competente
EU — France	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	ANSM / Autorité Compétente
EU — Deutschland	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, Frankreich Compliance@meetdandy.com	Reg: 989857065	BfArM / Zuständige Behörde
EU — Italia	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Parigi, Francia Compliance@meetdandy.com	Reg: 989857065	Min. della Salute / ISS
United Kingdom (UK)	DANDY LABS GB, LTD	5 New Street Square, London EC4A 3TW Compliance@meetdandy.com	Co. No.: 16873608	MHRA

NOTE

Contact details are subject to change. Always verify current information on the official product labelling or the manufacturer's website before reporting an incident. / Les coordonnées sont susceptibles de changer. / Angaben können sich ändern. / I recapiti sono soggetti a modifiche.

**Manufacturer / Hersteller /
Fabbicante / Fabricant /
Fabricante**

Zima International, Inc. Also DBA Dandy
1320 N. 300 W., Lehi, UT 84043, United States
compliance@meetdandy.com

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Approve	Kasha Lim	Supplier Quality Engineer	12-Jun-2026 01:58

* Datetime fields are displayed according to the system time zone: (GMT-04:00) Eastern Daylight Time (America/New_York)