INSTRUCTIONS FOR USE (IFU)

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MANUFACTURER

Manufacturer	Zima International, Inc. Also DBA Dandy
Address	1320 N. 300 W., Lehi, UT 84043, United States
Contact / Website	www.meetdandy.com
Complaints	complaints@meetdandy.com

DEVICE IDENTITY

Device Identity

Device Family	Inlay - Metal
GMDN / EMDN Code	38648
Product Code(s)	GLDHNSINA, GLDHNSINP, GLDNPSINP, GLDNWSINP, GLDNYSINP
Device Type	Custom-made dental device (CMD) / Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reuse	Reusable
Intended User	Prescriber: Dental professional / dental practice

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AUSTRALIA (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

Australia — Patient Matched Medical Device (PMMD)

GMDN / EMDN	38648
Product Code(s)	GLDHNSINA, GLDHNSINP, GLDNPSINP, GLDNWSINP, GLDNYSINP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
Australian Sponsor	Name: DANDY Labs AUS Pty Ltd Company No.: ABN: 33693955761 Address: Tower One - International Towers, Level 46, 100 Barangaroo Avenue, Sydney, New South Wales 2000, Australia
Intended User	Prescriber — Dental professional / dental practice
Device Type	Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

Dandy's Inlay/Onlay restorations are designed to restore worn, decayed, or broken dentition.

They:

Restore single, permanent teeth that are fractured, carious, or severely worn.

Restore the form and function of natural dentition.

Provide occlusal stability and maintain proper interarch relationships.

More conservative preparation, compared to Crowns and Bridges.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's restorations are fabricated using digital technology, enabling a precise fit and customization tailored to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function.
- Material: High-Noble: Rich Yellow (ArgenMill Au58), Noble White (ArgencoW), Non-Precious (ArgenMillCoCr)

[Full Metal Material Safety Data Sheets](#)

3) Contraindications

Inlays/Onlays should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, or oral lesions.

4) Warnings / Precautions / Potential Risks

WARNING

The use of fixed restorations may have potential risks, including failure, misfit, breakage, and recurrent caries.

****Warning – Sensitivity:**

This device contains a level of metal (including nickel). Patients with sensitivity or allergy to metals should be informed by the treating clinician of the presence of these compounds in the device prior to treatment. In rare cases, allergic reactions may occur in sensitized individuals.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in certain individuals due to individual variability in tissue response. Patients should be monitored post-placement, and the treating clinician should evaluate any signs of irritation, inflammation, or discomfort.

CAUTION

na

5) Cleaning and Care

Daily Cleaning

- Oral Hygiene:
 - Brushing:
 - Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.
 - Flossing:
 - Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas.
 - Mouthwash:
 - Alcohol-based mouthwashes tend to dry out the mouth and lead to increased plaque and tartar buildup.

Dietary and Habit Recommendations

- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods: where new cavities can form or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water will help keep your patient's mouth clean and prevent gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or biting on hard objects

6) Storage

Stored in specialized boxes with foam or membranes for protection, often made of ABS plastic. For patients who lose a crown, store it in a sealed bag or a small container with a small amount of water to keep it safe and moist until the dentist's visit.

7) Expected Life / Service

They offer exceptional longevity, typically lasting 20 years or more, often extending to 30 years or even a lifetime with excellent care, thanks to their unmatched durability, resistance to chipping, and precise fit.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy Australia entity using the contacts above, and
- Therapeutic Goods Administration (TGA)

In Australia, serious incidents involving medical devices must be reported to the TGA in accordance with the Therapeutic Goods Act 1989.

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

- Dandy asks that restorations that contain precious metals created by Dandy that need refabrication, be returned to us.
- Disposing of metal dental restorations involves either recycling them as precious metals (especially if gold alloy) through specialized refineries or scrap dealers for value, or general recycling if non-precious (like steel), but never in regular trash.
 - Precious Metal Refining: Many dental restorations contain valuable metals, making them a valuable asset for recycling.
 - Specialized Refiners: Some companies offer mail-in programs where you send in your scrap in prepaid, insured envelopes for assay and payment.
- How it Works: You send your restorations, they refine the metals, and pay you based on current market prices, often with insured shipping.
- For Non-Precious Metal Restorations
 - Dandy restorations that need refabrication, do not have to be returned to us.
 - Scrap Metal Dealers: various scrap metals, offering pickup or drop-off, though you'd need to confirm if they take small dental items.
 - General Recycling: Check with your local sanitation department for guidelines on disposing of bulk or metal items if it's not a dental office scenario.

EUROPEAN UNION (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

European Union — Custom-Made Device (CMD)

GMDN / EMDN	Q010206 - Dental Prostheses
Product Code(s)	GLDHNSINA, GLDHNSINP, GLDNPSINP, GLDNWSINP, GLDNYSINP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
EU authorized representative (EURP)	Name: Dandy Labs Europe Company No.: Registration No.: 989857065 Address: 3, boulevard de Sebastopol, 75001 Paris, France
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

Dandy's Inlay/Onlay restorations are designed to restore worn, decayed, or broken dentition.

They:

- Restore single, permanent teeth that are fractured, carious, or severely worn.
- Restore the form and function of natural dentition.
- Provide occlusal stability and maintain proper interarch relationships.
- More conservative preparation, compared to Crowns and Bridges.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's restorations are fabricated using digital technology, enabling a precise fit and customization tailored to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function.
- Material: High-Noble: Rich Yellow (ArgenMill Au58), Noble White (ArgencoW), Non-Precious (ArgenMillCoCr)

[Full Metal Material Safety Data Sheets](#)

3) Contraindications

Inlays/Onlays should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, or oral lesions.

4) Warnings / Precautions / Potential Risks

WARNING

The use of fixed restorations may have potential risks, including failure, misfit, breakage, and recurrent caries.

****Warning – Sensitivity:**

This device contains a level of metal (including nickel). Patients with sensitivity or allergy to metals should be informed by the treating clinician of the presence of these compounds in the device prior to treatment. In rare cases, allergic reactions may occur in sensitized individuals.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in certain individuals due to individual variability in tissue response. Patients should be monitored post-placement, and the treating clinician should evaluate any signs of irritation, inflammation, or discomfort.

CAUTION

na

5) Cleaning and Care

Daily Cleaning

- Oral Hygiene:
 - Brushing:
 - Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.
 - Flossing:
 - Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas.
 - Mouthwash:
 - Alcohol-based mouthwashes tend to dry out the mouth and lead to increased plaque and tartar buildup.

Dietary and Habit Recommendations

- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods: where new cavities can form or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water will help keep your patient's mouth clean and prevent gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or biting on hard objects

6) Storage

Stored in specialized boxes with foam or membranes for protection, often made of ABS plastic. For patients who lose a crown, store it in a sealed bag or a small container with a small amount of water to keep it safe and moist until the dentist's visit.

7) Expected Life / Service

They offer exceptional longevity, typically lasting 20 years or more, often extending to 30 years or even a lifetime with excellent care, thanks to their unmatched durability, resistance to chipping, and precise fit.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy EU entity using the contacts above, and corresponding local requirements
- The Competent Authority of your Member State, in accordance with Regulation (EU) 2017/745 (MDR).

Competent Authorities:

- France: Agence Nationale de Sécurité du Médicament et des Produits de Santé (ANSM)
- Spain : Spanish Agency for Medicines and Health Products (AEMPS)
- Germany : Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM) (BfArM)
- Italy : Ministero della Salute / Istituto Superiore di Sanità (ISS)
- Spain: Agencia Española de Medicamentos y Productos Sanitarios (AEMPS) for Spain
- Netherlands: Inspectie Gezondheidszorg en Jeugd (IGJ)
- Austria: Bundesamt für Sicherheit im Gesundheitswesen (BASG)
- Sweden: Läkemedelsverket
- Denmark: Lægemiddelstyrelsen
- Portugal: Autoridade Nacional do Medicamento e Produtos de Saúde (INFARMED)

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

- Dandy asks that restorations that contain precious metals created by Dandy that need refabrication, be returned to us.
- Disposing of metal dental restorations involves either recycling them as precious metals (especially if gold alloy) through specialized refineries or scrap dealers for value, or general recycling if non-precious (like steel), but never in regular trash.
 - Precious Metal Refining: Many dental restorations contain valuable metals, making them a valuable asset for recycling.
 - Specialized Refiners: Some companies offer mail-in programs where you send in your scrap in prepaid, insured envelopes for assay and payment.
- How it Works: You send your restorations, they refine the metals, and pay you based on current market prices, often with insured shipping.
- For Non-Precious Metal Restorations
 - Dandy restorations that need refabrication, do not have to be returned to us.
 - Scrap Metal Dealers: various scrap metals, offering pickup or drop-off, though you'd need to confirm if they take small dental items.
 - General Recycling: Check with your local sanitation department for guidelines on disposing of bulk or metal items if it's not a dental office scenario.

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UNITED KINGDOM (ENGLISH)

INSTRUCTIONS FOR USE (IFU)

United Kingdom — Custom-Made Device (CMD)

GMDN / EMDN	38648
Product Code(s)	GLDHNSINA, GLDHNSINP, GLDNPSINP, GLDNWSINP, GLDNYSINP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
UK Responsible Person (UKRP)	Name: DANDY LABS GB, LTD Company No.: Company Number: 16873608 Address: 5 New Street Square, London EC4A 3TW, United Kingdom
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Reusable

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- The Manufacturer and/or UK Responsible Person (UKRP) using the contacts above, and
- The UK Medicines and Healthcare products Regulatory Agency (MHRA) — www.gov.uk/mhra

In accordance with The Medical Devices Regulations 2002 (as amended).

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The table below lists the local Authorized Representative or Sponsor for each country / jurisdiction where this device is marketed. For incident reporting, contact the representative for your country.

La tabla siguiente enumera el Representante Autorizado o Patrocinador local para cada país. | Le tableau ci-dessous liste le Représentant Autorisé local pour chaque pays. | Die folgende Tabelle enthält die bevollmächtigten Vertreter je Land.

Jurisdiction	Authorised Rep / Sponsor	Registered Address / Contact Email	Reg. Number	Regulatory Body
Australia (AUS)	DANDY Labs AUS Pty Ltd	Tower One, Level 46, 100 Barangaroo Ave, Sydney NSW 2000 Compliance@meetdandy.com	ABN: 33693955761	TGA (Therapeutic Goods Administration)
EU — English	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	Competent Authority / EUDAMED
EU — España	Dandy Labs Europa	3, boulevard de Sebastopol, 75001 París, Francia Compliance@meetdandy.com	Reg: 989857065	AEMPS / Autoridad Competente
EU — France	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	ANSM / Autorité Compétente
EU — Deutschland	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, Frankreich Compliance@meetdandy.com	Reg: 989857065	BfArM / Zuständige Behörde
EU — Italia	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Parigi, Francia Compliance@meetdandy.com	Reg: 989857065	Min. della Salute / ISS
United Kingdom (UK)	DANDY LABS GB, LTD	5 New Street Square, London EC4A 3TW Compliance@meetdandy.com	Co. No.: 16873608	MHRA

NOTE

Contact details are subject to change. Always verify current information on the official product labelling or the manufacturer's website before reporting an incident. / Les coordonnées sont susceptibles de changer. / Angaben können sich ändern. / I recapiti sono soggetti a modifiche.

**Manufacturer / Hersteller /
Fabbicante / Fabricant /
Fabricante**

Zima International, Inc. Also DBA Dandy
1320 N. 300 W., Lehi, UT 84043, United States
compliance@meetdandy.com

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Approve	Kasha Lim	Supplier Quality Engineer	12-Jun-2026 01:53

* Datetime fields are displayed according to the system time zone: (GMT-04:00) Eastern Daylight Time (America/New_York)