INSTRUCTIONS FOR USE (IFU)

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MANUFACTURER

Manufacturer	Zima International, Inc. Also DBA Dandy
Address	1320 N. 300 W., Lehi, UT 84043, United States
Contact / Website	www.meetdandy.com
Complaints	complaints@meetdandy.com

DEVICE IDENTITY

Device Identity

Device Family	Complete Upper Denture
GMDN / EMDN Code	38588
Product Code(s)	ACRFDENT, DENTCOPY, DENTFLRLN, DENTMDTRY, DENTSMOTRY, DENTSPRPR, FDAOMF, FDAOMM, FDAOOCG, MILLDCR, MILLDENT, MILLDIMFD
Device Type	Custom-made dental device (CMD) / Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reuse	Reusable
Intended User	Prescriber: Dental professional / dental practice

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AUSTRALIA (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

Australia — Patient Matched Medical Device (PMMD)

GMDN / EMDN	38588
Product Code(s)	ACRFDENT, DENTCOPY, DENTFLRLN, DENTMDTRY, DENTSMOTRY, DENTSPRPR, FDAOMF, FDAOMM, FDAOOCG, MILLDCR, MILLDENT, MILLDIMFD
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
Australian Sponsor	Name: DANDY Labs AUS Pty Ltd Company No.: ABN: 33693955761 Address: Tower One - International Towers, Level 46, 100 Barangaroo Avenue, Sydney, New South Wales 2000, Australia
Intended User	Prescriber — Dental professional / dental practice
Device Type	Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

Dandy's full dentures are designed/fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of highly durable and biocompatible materials, ensuring long-term comfort and function.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's Dentures are removable prostheses intended for the replacement of missing natural teeth and associated oral structures. They are indicated for fully edentulous and partially edentulous patients requiring prosthetic restoration.
- Material:

HIPA : High-impact resistance, lifelike esthetics

199 Resin : Lightweight, digitally accurate

PMMA : Seamless monolithic finish

[Full Denture MSDS](#)

3) Contraindications

Do not use in patients with:

- Uncontrolled periodontal disease
- Severe residual ridge resorption
- Untreated oral lesions
- Known hypersensitivity to acrylic or PMMA-based materials

Avoid in patients allergic to methacrylate monomers.

4) Warnings / Precautions / Potential Risks

WARNING

Gingival irritation, mucosal soreness, ulceration
 Poor adaptation may cause instability or trauma
 Allergic response to base or teeth materials

CAUTION

NA

5) Cleaning and Care

To ensure proper use of final dentures, dental professionals and patients should follow these steps:

Insertion Protocol:

- Clean oral tissues with a soft-bristle brush or rinse
- Hold the denture by the edges; insert using a gentle rocking motion
- If it is a partial denture, gently place it in the mouth and use your fingers to guide it into position.
- Avoid using excessive force or pushing the partial denture too far.
- Instruct the patient to bite and swallow to seat
- If it is a partial denture, instruct the patient to gently bite down on the partial denture to ensure proper fit and stability.
- Use adhesive if retention is insufficient
- Use carbon articulation paper to adjust any heavy occlusion points.

Cleaning and Care **

DAILY CLEANING

- Dandy recommends cleaning your dentures daily before and after every use. Immediately after removal, use warm water (not to exceed 45°C/113°F) and dishwashing soap to remove remaining saliva and debris from the device.
- Only use soft, antibacterial dish soap (liquid)
- Do not use flavored or scented soaps
- Do not use soaps with heavy dyes and colors

USING CLEANING AGENTS

- For periodic debris removal and maintenance, use a soft-bristle brush to clean the inside of your appliance. The denture should be cleaned with antibacterial denture cleaning solutions like Polident and Efferdent. These cleaners are designed for use in brief sessions of 10–15 minutes. Always make sure to follow the manufacturer's instructions for proper use.

Do not use:

- Avoid immersing the denture in, or cleaning it with, any of the following substances: sodium hypochlorite (chlorine bleach), hydrogen peroxide, mouthwash, or alcohol products. Using these can lead to discoloration and damage to the material.

6) Storage

When not in use, store final dentures in a clean, dry place. Avoid exposing the dentures to excessive heat or pressure, which may cause warping or other damage.

7) Expected Life / Service

Final dentures are designed to last for several years with proper care and maintenance. Patients should contact their treating dentist/practice for evaluation and repair if dentures become damaged or lose their fit.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy Australia entity using the contacts above, and
- Therapeutic Goods Administration (TGA)

In Australia, serious incidents involving medical devices must be reported to the TGA in accordance with the Therapeutic Goods Act 1989.

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

- Denture disposal typically involves handling it as a regulated biohazardous waste or general waste, often through specialized handlers for incineration or secure landfill disposal.
- Always avoid putting them in regular trash without checking local rules.
- Biohazard/Infectious Waste: Treat used or unclaimed dentures as biohazardous due to saliva/blood; use red bags and authorized carriers.
- Incineration: A common method for final destruction.
- Metal Recovery: If the denture has precious metals (gold/silver), return it to a dental lab or scrap refiner for recycling, as it can be profitable and prevents pollution.
- Confidentiality: Redact or destroy patient info on lab boxes and bases before disposal.

EUROPEAN UNION (ENGLISH)

INSTRUCTIONS FOR USE (IFU)

European Union — Custom-Made Device (CMD)

GMDN / EMDN	Q01020601: Mobile Dental Prostheses
Product Code(s)	ACRFDENT, DENTCOPY, DENTFLRLN, DENTMDTRY, DENTSMOTRY, DENTSPRPR, FDAOMF, FDAOMM, FDAOOCG, MILDCR, MILLDENT, MILLDIMFD
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
EU authorized representative (EURP)	Name: Dandy Labs Europe Company No.: Registration No.: 989857065 Address: 3, boulevard de Sebastopol, 75001 Paris, France
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Reusable

1) Intended Purpose (Intended Use)

Dandy's full dentures are designed/fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of highly durable and biocompatible materials, ensuring long-term comfort and function.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's Dentures are removable prostheses intended for the replacement of missing natural teeth and associated oral structures. They are indicated for fully edentulous and partially edentulous patients requiring prosthetic restoration.
- Material:

HIPA : High-impact resistance, lifelike esthetics

199 Resin : Lightweight, digitally accurate

PMMA : Seamless monolithic finish

[Full Denture MSDS](#)

3) Contraindications

Do not use in patients with:

- Uncontrolled periodontal disease
- Severe residual ridge resorption
- Untreated oral lesions
- Known hypersensitivity to acrylic or PMMA-based materials

Avoid in patients allergic to methacrylate monomers.

4) Warnings / Precautions / Potential Risks

WARNING

- Gingival irritation, mucosal soreness, ulceration
- Poor adaptation may cause instability or trauma
- Allergic response to base or teeth materials

CAUTION

NA

5) Cleaning and Care

To ensure proper use of final dentures, dental professionals and patients should follow these steps:

Insertion Protocol:

- Clean oral tissues with a soft-bristle brush or rinse
- Hold the denture by the edges; insert using a gentle rocking motion
- If it is a partial denture, gently place it in the mouth and use your fingers to guide it into position.
- Avoid using excessive force or pushing the partial denture too far.
- Instruct the patient to bite and swallow to seat
- If it is a partial denture, instruct the patient to gently bite down on the partial denture to ensure proper fit and stability.
- Use adhesive if retention is insufficient
- Use carbon articulation paper to adjust any heavy occlusion points.

Cleaning and Care **

DAILY CLEANING

- Dandy recommends cleaning your dentures daily before and after every use. Immediately after removal, use warm water (not to exceed 45°C/113°F) and dishwashing soap to remove remaining saliva and debris from the device.
- Only use soft, antibacterial dish soap (liquid)
- Do not use flavored or scented soaps
- Do not use soaps with heavy dyes and colors

USING CLEANING AGENTS

- For periodic debris removal and maintenance, use a soft-bristle brush to clean the inside of your appliance. The denture should be cleaned with antibacterial denture cleaning solutions like Polident and Efferdent. These cleaners are designed for use in brief sessions of 10–15 minutes. Always make sure to follow the manufacturer's instructions for proper use.

Do not use:

- Avoid immersing the denture in, or cleaning it with, any of the following substances: sodium hypochlorite (chlorine bleach), hydrogen peroxide, mouthwash, or alcohol products. Using these can lead to discoloration and damage to the material.

6) Storage

When not in use, store final dentures in a clean, dry place. Avoid exposing the dentures to excessive heat or pressure, which may cause warping or other damage.

7) Expected Life / Service

Final dentures are designed to last for several years with proper care and maintenance. Patients should contact their treating dentist/practice for evaluation and repair if dentures become damaged or lose their fit.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy EU entity using the contacts above, and corresponding local requirements
- The Competent Authority of your Member State, in accordance with Regulation (EU) 2017/745 (MDR).

Competent Authorities:

- France: Agence Nationale de Sécurité du Médicament et des Produits de Santé (ANSM)
- Spain : Spanish Agency for Medicines and Health Products (AEMPS)
- Germany : Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM) (BfArM)
- Italy : Ministero della Salute / Istituto Superiore di Sanità (ISS)
- Spain: Agencia Española de Medicamentos y Productos Sanitarios (AEMPS) for Spain
- Netherlands: Inspectie Gezondheidszorg en Jeugd (IGJ)
- Austria: Bundesamt für Sicherheit im Gesundheitswesen (BASG)
- Sweden: Läkemedelsverket
- Denmark: Lægemiddelstyrelsen
- Portugal: Autoridade Nacional do Medicamento e Produtos de Saúde (INFARMED)

INFO

For the full contact information of the local Authorized Representative, see: Authorized Representatives — at the end of the document.

9) Disposal

- Denture disposal typically involves handling it as a regulated biohazardous waste or general waste, often through specialized handlers for incineration or secure landfill disposal.
- Always avoid putting them in regular trash without checking local rules.
- Biohazard/Infectious Waste: Treat used or unclaimed dentures as biohazardous due to saliva/blood; use red bags and authorized carriers.
- Incineration: A common method for final destruction.
- Metal Recovery: If the denture has precious metals (gold/silver), return it to a dental lab or scrap refiner for recycling, as it can be profitable and prevents pollution.
- Confidentiality: Redact or destroy patient info on lab boxes and bases before disposal.

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UNITED KINGDOM (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

United Kingdom — Custom-Made Device (CMD)

GMDN / EMDN	38588
Product Code(s)	ACRFDENT, DENTCOPY, DENTFLRLN, DENTMDTRY, DENTSMOTRY, DENTSPRPR, FDAOMF, FDAOMM, FDAOOCG, MILLDCR, MILLDENT, MILLDIMFD
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
UK Responsible Person (UKRP)	Name: DANDY LABS GB, LTD Company No.: Company Number: 16873608 Address: 5 New Street Square, London EC4A 3TW, United Kingdom
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Reusable

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- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's Dentures are removable prostheses intended for the replacement of missing natural teeth and associated oral structures. They are indicated for fully edentulous and partially edentulous patients requiring prosthetic restoration.
- Material:
 - HIPA : High-impact resistance, lifelike esthetics
 - 199 Resin : Lightweight, digitally accurate
 - PMMA : Seamless monolithic finish

[Full Denture MSDS](#)

3) Contraindications

Do not use in patients with:

- Uncontrolled periodontal disease
- Severe residual ridge resorption
- Untreated oral lesions
- Known hypersensitivity to acrylic or PMMA-based materials
- Avoid in patients allergic to methacrylate monomers.

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- The Manufacturer and/or UK Responsible Person (UKRP) using the contacts above, and
- The UK Medicines and Healthcare products Regulatory Agency (MHRA) — www.gov.uk/mhra

In accordance with The Medical Devices Regulations 2002 (as amended).

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The table below lists the local Authorized Representative or Sponsor for each country / jurisdiction where this device is marketed. For incident reporting, contact the representative for your country.

La tabla siguiente enumera el Representante Autorizado o Patrocinador local para cada país. | Le tableau ci-dessous liste le Représentant Autorisé local pour chaque pays. | Die folgende Tabelle enthält die bevollmächtigten Vertreter je Land.

Jurisdiction	Authorised Rep / Sponsor	Registered Address / Contact Email	Reg. Number	Regulatory Body
Australia (AUS)	DANDY Labs AUS Pty Ltd	Tower One, Level 46, 100 Barangaroo Ave, Sydney NSW 2000 Compliance@meetdandy.com	ABN: 33693955761	TGA (Therapeutic Goods Administration)
EU — English	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	Competent Authority / EUDAMED
EU — España	Dandy Labs Europa	3, boulevard de Sebastopol, 75001 París, Francia Compliance@meetdandy.com	Reg: 989857065	AEMPS / Autoridad Competente
EU — France	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	ANSM / Autorité Compétente
EU — Deutschland	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, Frankreich Compliance@meetdandy.com	Reg: 989857065	BfArM / Zuständige Behörde
EU — Italia	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Parigi, Francia Compliance@meetdandy.com	Reg: 989857065	Min. della Salute / ISS
United Kingdom (UK)	DANDY LABS GB, LTD	5 New Street Square, London EC4A 3TW Compliance@meetdandy.com	Co. No.: 16873608	MHRA

NOTE

Contact details are subject to change. Always verify current information on the official product labelling or the manufacturer's website before reporting an incident. / Les coordonnées sont susceptibles de changer. / Angaben können sich ändern. / I recapiti sono soggetti a modifiche.

**Manufacturer / Hersteller /
Fabbicante / Fabricant /
Fabricante**

Zima International, Inc. Also DBA Dandy
1320 N. 300 W., Lehi, UT 84043, United States
compliance@meetdandy.com

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Approve	Kasha Lim	Supplier Quality Engineer	12-Jun-2026 01:19

* Datetime fields are displayed according to the system time zone: (GMT-04:00) Eastern Daylight Time (America/New_York)