INSTRUCTIONS FOR USE (IFU)

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MANUFACTURER

Manufacturer	Zima International, Inc. Also DBA Dandy
Address	1320 N. 300 W., Lehi, UT 84043, United States
Contact / Website	www.meetdandy.com
Complaints	complaints@meetdandy.com

DEVICE IDENTITY

Device Identity

Device Family	Bridge - Polymer
GMDN / EMDN Code	37542
Product Code(s)	PMMASBRA, PMMASBRP
Device Type	Custom-made dental device (CMD) / Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reuse	Single-use
Intended User	Prescriber: Dental professional / dental practice

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AUSTRALIA (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

Australia — Patient Matched Medical Device (PMMD)

GMDN / EMDN	37542
Product Code(s)	PMMASBRA, PMMASBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
Australian Sponsor	Name: DANDY Labs AUS Pty Ltd Company No.: ABN: 33693955761 Address: Tower One - International Towers, Level 46, 100 Barangaroo Avenue, Sydney, New South Wales 2000, Australia
Intended User	Prescriber — Dental professional / dental practice
Device Type	Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Single-use

1) Intended Purpose (Intended Use)

Dandy's temporary bridges are made to provisionally restore worn, carious, broken, and aesthetically displeasing dentition.

They:

- Replace one or more missing permanent teeth (bridges) by spanning the edentulous space and being supported by natural teeth or dental implants
- Restore the form, function, and esthetics of natural dentition.
- Provide occlusal stability and maintain proper interarch relationships.
- Provide a preview of the final restoration.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's temporary bridges (PMMA) are fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function. PMMA, which stands for polymethyl methacrylate, is not intended for short-term use.
- Material: Argen PMMA

[Temporary Crown and Bridge Material Safety Data Sheet](#)

3) Contraindications

Temporary crowns should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, active caries, or oral lesions.

4) Warnings / Precautions / Potential Risks

WARNING

The use of temporary bridges may have potential risks, including failure, misfit, breakage, and recurrent caries.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in specific individuals due to individual variability in tissue response. Patients should be monitored after placement, and the treating clinician should assess any signs of irritation, inflammation, or discomfort.

CAUTION

na

5) Cleaning and Care

Daily Cleaning

Oral Hygiene:

Brushing:

- Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.

Flossing:

- Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas (for bridges). Care must be taken to prevent the dislodging of the restoration.

Mouthwash:

- Alcohol-based mouthwashes tend to dry out the mouth, leading to increased plaque and tartar buildup.
- Dietary and Habit Recommendations
- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods, as they can form new cavities or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water helps keep your patient's mouth clean and prevents gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or biting on hard objects.

6) Storage

Requires a cool, dry place, protected from direct sunlight, extreme heat (above 77°F/25°C), moisture, dust, and contamination, as these conditions can degrade the material or compromise its integrity, affecting temporary use and ultimately impacting shelf life and performance.

7) Expected Life / Service

PMMA (polymethyl methacrylate) dental bridges generally last 3 to 5 years, but often require more frequent attention than other materials, depending heavily on oral hygiene, habits (avoiding hard/sticky foods), bridge design, and location (front vs. back teeth).

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy Australia entity using the contacts above, and
- Therapeutic Goods Administration (TGA)

In Australia, serious incidents involving medical devices must be reported to the TGA in accordance with the Therapeutic Goods Act 1989.

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

- Disposal of PMMA (Polymethyl Methacrylate) dental restorations involves recycling, often through specialized dental waste handlers who can chemically break it down into its monomer (MMA) or manage it as plastic. However, it should not be placed in the general trash due to environmental concerns.
- Fully cured PMMA might also be treated like general plastic waste if not mixed with metals or hazardous components.
- No Regular Trash: Do not dispose of PMMA dental waste in the regular garbage or medical waste streams.
- Hazardous Components: If mixed with solvents or other hazardous materials, it requires specific hazardous waste handling.

EUROPEAN UNION (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

European Union — Custom-Made Device (CMD)

GMDN / EMDN	Q010206- Dental Prostheses
Product Code(s)	PMMASBRA, PMMASBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
EU authorized representative (EURP)	Name: Dandy Labs Europe Company No.: Registration No.: 989857065 Address: 3, boulevard de Sebastopol, 75001 Paris, France
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile (if applicable)
Single-use / Reusable	Single-use

1) Intended Purpose (Intended Use)

Dandy's temporary bridges are made to provisionally restore worn, carious, broken, and aesthetically displeasing dentition.

They:

- Replace one or more missing permanent teeth (bridges) by spanning the edentulous space and being supported by natural teeth or dental implants
- Restore the form, function, and esthetics of natural dentition.
- Provide occlusal stability and maintain proper interarch relationships.
- Provide a preview of the final restoration.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's temporary bridges (PMMA) are fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function. PMMA, which stands for polymethyl methacrylate, is not intended for short-term use.
- Material: Argen PMMA

[Temporary Crown and Bridge Material Safety Data Sheet](#)

3) Contraindications

Temporary crowns should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, active caries, or oral lesions.

4) Warnings / Precautions / Potential Risks

WARNING

The use of temporary bridges may have potential risks, including failure, misfit, breakage, and recurrent caries.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in specific individuals due to individual variability in tissue response. Patients should be monitored after placement, and the treating clinician should assess any signs of irritation, inflammation, or discomfort.

CAUTION

na

5) Cleaning and Care

Daily Cleaning

Oral Hygiene:

Brushing:

- Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.

Flossing:

- Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas (for bridges). Care must be taken to prevent the dislodging of the restoration.

Mouthwash:

- Alcohol-based mouthwashes tend to dry out the mouth, leading to increased plaque and tartar buildup.
- Dietary and Habit Recommendations
- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods, as they can form new cavities or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water helps keep your patient's mouth clean and prevents gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or biting on hard objects.

6) Storage

Requires a cool, dry place, protected from direct sunlight, extreme heat (above 77°F/25°C), moisture, dust, and contamination, as these conditions can degrade the material or compromise its integrity, affecting temporary use and ultimately impacting shelf life and performance.

7) Expected Life / Service

PMMA (polymethyl methacrylate) dental bridges generally last 3 to 5 years, but often require more frequent attention than other materials, depending heavily on oral hygiene, habits (avoiding hard/sticky foods), bridge design, and location (front vs. back teeth).

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy EU entity using the contacts above, and corresponding local requirements
- The Competent Authority of your Member State, in accordance with Regulation (EU) 2017/745 (MDR).

Competent Authorities:

- France: Agence Nationale de Sécurité du Médicament et des Produits de Santé (ANSM)
- Spain : Spanish Agency for Medicines and Health Products (AEMPS)
- Germany : Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM) (BfArM)
- Italy : Ministero della Salute / Istituto Superiore di Sanità (ISS)
- Spain: Agencia Española de Medicamentos y Productos Sanitarios (AEMPS) for Spain
- Netherlands: Inspectie Gezondheidszorg en Jeugd (IGJ)
- Austria: Bundesamt für Sicherheit im Gesundheitswesen (BASG)
- Sweden: Läkemedelsverket
- Denmark: Lægemiddelstyrelsen
- Portugal: Autoridade Nacional do Medicamento e Produtos de Saúde (INFARMED)

INFO

For the full contact information of the local Authorized Representative, see: Authorized Representatives — at the end of the document.

9) Disposal

- Disposal of PMMA (Polymethyl Methacrylate) dental restorations involves recycling, often through specialized dental waste handlers who can chemically break it down into its monomer (MMA) or manage it as plastic. However, it should not be placed in the general trash due to environmental concerns.
- Fully cured PMMA might also be treated like general plastic waste if not mixed with metals or hazardous components.
- No Regular Trash: Do not dispose of PMMA dental waste in the regular garbage or medical waste streams.
- Hazardous Components: If mixed with solvents or other hazardous materials, it requires specific hazardous waste handling.

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UNITED KINGDOM (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

United Kingdom — Custom-Made Device (CMD)

GMDN / EMDN	37542
Product Code(s)	PMMASBRA, PMMASBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
UK Responsible Person (UKRP)	Name: DANDY LABS GB, LTD Company No.: Company Number: 16873608 Address: 5 New Street Square, London EC4A 3TW, United Kingdom
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Single-use

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- The Manufacturer and/or UK Responsible Person (UKRP) using the contacts above, and
- The UK Medicines and Healthcare products Regulatory Agency (MHRA) — www.gov.uk/mhra

In accordance with The Medical Devices Regulations 2002 (as amended).

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The table below lists the local Authorized Representative or Sponsor for each country / jurisdiction where this device is marketed. For incident reporting, contact the representative for your country.

La tabla siguiente enumera el Representante Autorizado o Patrocinador local para cada país. | Le tableau ci-dessous liste le Représentant Autorisé local pour chaque pays. | Die folgende Tabelle enthält die bevollmächtigten Vertreter je Land.

Jurisdiction	Authorised Rep / Sponsor	Registered Address / Contact Email	Reg. Number	Regulatory Body
Australia (AUS)	DANDY Labs AUS Pty Ltd	Tower One, Level 46, 100 Barangaroo Ave, Sydney NSW 2000 Compliance@meetdandy.com	ABN: 33693955761	TGA (Therapeutic Goods Administration)
EU — English	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	Competent Authority / EUDAMED
EU — España	Dandy Labs Europa	3, boulevard de Sebastopol, 75001 Paris, Francia Compliance@meetdandy.com	Reg: 989857065	AEMPS / Autoridad Competente
EU — France	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	ANSM / Autorité Compétente
EU — Deutschland	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, Frankreich Compliance@meetdandy.com	Reg: 989857065	BfArM / Zuständige Behörde
EU — Italia	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Parigi, Francia Compliance@meetdandy.com	Reg: 989857065	Min. della Salute / ISS
United Kingdom (UK)	DANDY LABS GB, LTD	5 New Street Square, London EC4A 3TW Compliance@meetdandy.com	Co. No.: 16873608	MHRA

NOTE

Contact details are subject to change. Always verify current information on the official product labelling or the manufacturer's website before reporting an incident. / Les coordonnées sont susceptibles de changer. / Angaben können sich ändern. / I recapiti sono soggetti a modifiche.

Manufacturer / Hersteller / Fabbricante / Fabricant / Fabricante	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043, United States compliance@meetdandy.com
--	---

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Approve	Kasha Lim	Supplier Quality Engineer	12-Jun-2026 01:12

* Datetime fields are displayed according to the system time zone: (GMT-04:00) Eastern Daylight Time (America/New_York)