INSTRUCTIONS FOR USE (IFU)

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MANUFACTURER

Manufacturer	Zima International, Inc. Also DBA Dandy
Address	1320 N. 300 W., Lehi, UT 84043, United States
Contact / Website	www.meetdandy.com
Complaints	complaints@meetdandy.com

DEVICE IDENTITY

Device Identity

Device Family	Bridge - Ceramic
GMDN / EMDN Code	38615
Product Code(s)	EMXLPSBRA, EMXSBRA, EMXSBRP, PFZSBRA, PFZSBRP, ZIRHTSBRA, ZIRHTSBRP, ZIRMOSBRA, ZIRMOSBRP
Device Type	Custom-made dental device (CMD) / Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reuse	Single-use
Intended User	Prescriber: Dental professional / dental practice

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AUSTRALIA (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

Australia — Patient Matched Medical Device (PMMD)

GMDN / EMDN	38615
Product Code(s)	EMXLPSBRA, EMXSBR, EMXSBRP, PFZSBRA, PFZSBRP, ZIRHTSBRA, ZIRHTSBRP, ZIRMOSBRA, ZIRMOSBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
Australian Sponsor	Name: DANDY Labs AUS Pty Ltd Company No.: ABN: 33693955761 Address: Tower One - International Towers, Level 46, 100 Barangaroo Avenue, Sydney, New South Wales 2000, Australia
Intended User	Prescriber — Dental professional / dental practice
Device Type	Patient Matched Medical Device (PMMD)
Sterility	Not supplied sterile
Single-use / Reusable	Single-use

1) Intended Purpose (Intended Use)

Dandy's ceramic bridges are made to restore worn, carious, broken, and aesthetically displeasing dentition.

They:

- Replace one or more missing permanent teeth (bridges) by spanning the edentulous space and being supported by natural teeth or dental implants
- Restore the form, function, and esthetics of natural dentition.
- Provide occlusal stability and maintain proper interarch relationships.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's ceramic bridges are fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function.
- Material: Zirconia Monolithic (3Y & 4Y), Porcelain-Fused-To-Zirconia (PFZ), E.Max (full contour), E.Max (layered)

[Zirconia Material Safety Data Sheet](#)

[PFZ Material Safety Data Sheet](#)

Full Contour E.MAX Material Safety Data Sheet

3) Contraindications

Bridges should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, active caries, or oral lesions.

4) Warnings / Precautions / Potential Risks

⚠ WARNING

The use of fixed restorations may have potential risks, including failure, misfit, breakage, and recurrent caries.

****Warning – Sensitivity:**

This device contains trace levels of cobalt and chromium oxides used as colorants in the ceramic material. The treating clinician should inform patients with sensitivity or allergy to cobalt or chromium of the presence of these compounds in the device before treatment. In rare cases, allergic reactions can occur in individuals who are sensitized.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in certain individuals due to individual variability in tissue response. Patients should be monitored post-placement, and the treating clinician should evaluate any signs of irritation, inflammation, or discomfort

⚠ CAUTION

na

5) Cleaning and Care

Daily Cleaning**Oral Hygiene:****Brushing:**

- Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.

Flossing:

- Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas (for bridges).

Mouthwash:

- Alcohol-based mouthwashes tend to dry out the mouth, leading to increased plaque and tartar buildup.
- Dietary and Habit Recommendations
- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods: where new cavities can form or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water will help keep your patient's mouth clean and prevent gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or bite on hard objects.

6) Storage

Use dedicated dental crown/veneer boxes with soft foam or silicone inserts to prevent chipping, scratching, or movement, ensuring they are kept clean, dry, and protected during transport or while awaiting placement in a clinic or lab setting..

7) Expected Life / Service

Ceramic dental restorations (crowns, fillings, veneers, and bridges) generally last 10 to 20 years, with modern materials like zirconia potentially exceeding this lifespan. However, lifespans vary significantly based on care, location (back teeth wear faster), and type (implants are more permanent than crowns/bridges on top). Excellent oral hygiene (brushing, flossing) and avoiding hard foods or teeth grinding (bruxism) are key to maximizing longevity, though some chipping or wear is possible.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy Australia entity using the contacts above, and
- Therapeutic Goods Administration (TGA)

In Australia, serious incidents involving medical devices must be reported to the TGA in accordance with the Therapeutic Goods Act 1989.

INFO

For full contact details of the local Authorized Representative, refer to: Authorized Representatives — end of this document.

9) Disposal

- All-ceramic (porcelain) restorations are often disposed of as regular solid waste or taken to specific recycling centers that accept mixed ceramics/construction debris, as they're challenging to recycle like typical items.
- Household Trash: Place in your regular garbage bin, as they don't break down in standard recycling
- Always check with your local municipality or waste hauler for specific rules; however, ceramic items are generally not accepted in standard curbside recycling bins.
- Recycling Centers: Take them to a local recycling center that accepts mixed ceramics, concrete, or construction debris, as they can sometimes be repurposed.
- If the restoration has been in the patient's mouth or has come into contact with saliva, discard it as biomedical waste.

EUROPEAN UNION (ENGLISH)

INSTRUCTIONS FOR USE (IFU)

European Union — Custom-Made Device (CMD)

GMDN / EMDN	Q010206- Dental Prostheses
Product Code(s)	EMXLPSBRA, EMXS BRA, EMXSBRP, PFZSBRA, PFZSBRP, ZIRHTSBRA, ZIRHTSBRP, ZIRMOSBRA, ZIRMOSBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
EU Authorized Representative (EURP)	Name: Dandy Labs Europe Company No.: Registration No.: 989857065 Address: 3, boulevard de Sebastopol, 75001 Paris, France
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Single-use

1) Intended Purpose (Intended Use)

Dandy's ceramic bridges are made to restore worn, carious, broken, and aesthetically displeasing dentition.

They:

Replace one or more missing permanent teeth (bridges) by spanning the edentulous space and being supported by natural teeth or dental implants

Restore the form, function, and esthetics of natural dentition.

Provide occlusal stability and maintain proper interarch relationships.

2) Device Description / Key Specifications

- Custom-made, patient-specific device manufactured per dental professional prescription and digital input data.
- Configuration: Dandy's ceramic bridges are fabricated using digital technology, allowing for a precise fit and customization to the patient's specific needs. They are made of a highly durable and biocompatible material, ensuring long-term comfort and function.
- Material: Zirconia Monolithic (3Y & 4Y), Porcelain-Fused-To-Zirconia (PFZ), E.Max (full contour), E.Max (layered)

[Zirconia Material Safety Data Sheet](#)

[PFZ Material Safety Data Sheet](#)

Full Contour E.MAX Material Safety Data Sheet

3) Contraindications

Bridges should be used with caution in patients with uncontrolled periodontal disease, severe bone resorption, active caries, or oral lesions.

4) Warnings / Precautions / Potential Risks

WARNING

The use of fixed restorations may have potential risks, including failure, misfit, breakage, and recurrent caries.

****Warning – Sensitivity:**

This device contains trace levels of cobalt and chromium oxides used as colorants in the ceramic material. The treating clinician should inform patients with sensitivity or allergy to cobalt or chromium of the presence of these compounds in the device before treatment. In rare cases, allergic reactions can occur in individuals who are sensitized.

***Warning – Irritation:**

Localized irritation of surrounding oral tissues may occur in certain individuals due to individual variability in tissue response. Patients should be monitored post-placement, and the treating clinician should evaluate any signs of irritation, inflammation, or discomfort

CAUTION

na

5) Cleaning and Care

Daily Cleaning

Oral Hygiene:

Brushing:

- Recommended brushing teeth twice a day, using a soft-bristled toothbrush and a fluoride toothpaste.

Flossing:

- Recommend daily flossing to remove plaque and debris from between the teeth and around the fixed restoration. A floss threader may be needed for hard-to-reach areas (for bridges).

Mouthwash:

- Alcohol-based mouthwashes tend to dry out the mouth, leading to increased plaque and tartar buildup.
- Dietary and Habit Recommendations
- Avoid abrasive foods: Limit hard, crunchy, or sticky foods that can damage the restoration or cause it to loosen.
- Avoid sugary foods: where new cavities can form or cause premature decay of the restorative margins.
- Stay hydrated: Drinking plenty of water will help keep your patient's mouth clean and prevent gum disease.
- Avoid bad habits: Refrain from using teeth to open packages or bite on hard objects.

6) Storage

Use dedicated dental crown/veneer boxes with soft foam or silicone inserts to prevent chipping, scratching, or movement, ensuring they are kept clean, dry, and protected during transport or while awaiting placement in a clinic or lab setting..

7) Expected Life / Service

Ceramic dental restorations (crowns, fillings, veneers, and bridges) generally last 10 to 20 years, with modern materials like zirconia potentially exceeding this lifespan. However, lifespans vary significantly based on care, location (back teeth wear faster), and type (implants are more permanent than crowns/bridges on top). Excellent oral hygiene (brushing, flossing) and avoiding hard foods or teeth grinding (bruxism) are key to maximizing longevity, though some chipping or wear is possible.

8) Incident / Complaint Reporting

Report suspected serious incidents associated with this device to:

- The Manufacturer and/or Dandy EU entity using the contacts above, and corresponding local requirements
- The Competent Authority of your Member State, in accordance with Regulation (EU) 2017/745 (MDR).

Competent Authorities:

- France: Agence Nationale de Sécurité du Médicament et des Produits de Santé (ANSM)
- Spain : Spanish Agency for Medicines and Health Products (AEMPS)
- Germany : Bundesinstitut für Arzneimittel und Medizinprodukte (BfArM) (BfArM)
- Italy : Ministero della Salute / Istituto Superiore di Sanità (ISS)
- Spain: Agencia Española de Medicamentos y Productos Sanitarios (AEMPS) for Spain
- Netherlands: Inspectie Gezondheidszorg en Jeugd (IGJ)
- Austria: Bundesamt für Sicherheit im Gesundheitswesen (BASG)
- Sweden: Läkemedelsverket
- Denmark: Lægemiddelstyrelsen
- Portugal: Autoridade Nacional do Medicamento e Produtos de Saúde (INFARMED)

INFO

For the full contact information of the local Authorized Representative, see: Authorized Representatives — at the end of the document.

9) Disposal

- All-ceramic (porcelain) restorations are often disposed of as regular solid waste or taken to specific recycling centers that accept mixed ceramics/construction debris, as they're challenging to recycle like typical items.
- Household Trash: Place in your regular garbage bin, as they don't break down in standard recycling
- Always check with your local municipality or waste hauler for specific rules; however, ceramic items are generally not accepted in standard curbside recycling bins.
- Recycling Centers: Take them to a local recycling center that accepts mixed ceramics, concrete, or construction debris, as they can sometimes be repurposed.

-
- If the restoration has been in the patient's mouth or has come into contact with saliva, discard it as biomedical waste.

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UNITED KINGDOM (ENGLISH)**INSTRUCTIONS FOR USE (IFU)**

United Kingdom — Custom-Made Device (CMD)

GMDN / EMDN	38615
Product Code(s)	EMXLPSBRA, EMXSBRA, EMXSBRP, PFZSBRA, PFZSBRP, ZIRHTSBRA, ZIRHTSBRP, ZIRMOSBRA, ZIRMOSBRP
Manufacturer	Zima International, Inc. Also DBA Dandy 1320 N. 300 W., Lehi, UT 84043 United States
UK Responsible Person (UKRP)	Name: DANDY LABS GB, LTD Company No.: Company Number: 16873608 Address: 5 New Street Square, London EC4A 3TW, United Kingdom
Intended User	Prescriber — Dental professional / dental practice
Device Type	Custom-made dental device (CMD)
Sterility	Not supplied sterile
Single-use / Reusable	Single-use

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- The Manufacturer and/or UK Responsible Person (UKRP) using the contacts above, and
- The UK Medicines and Healthcare products Regulatory Agency (MHRA) — www.gov.uk/mhra

In accordance with The Medical Devices Regulations 2002 (as amended).

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The table below lists the local Authorized Representative or Sponsor for each country / jurisdiction where this device is marketed. For incident reporting, contact the representative for your country.

La tabla siguiente enumera el Representante Autorizado o Patrocinador local para cada país. | Le tableau ci-dessous liste le Représentant Autorisé local pour chaque pays. | Die folgende Tabelle enthält die bevollmächtigten Vertreter je Land.

Jurisdiction	Authorised Rep / Sponsor	Registered Address / Contact Email	Reg. Number	Regulatory Body
Australia (AUS)	DANDY Labs AUS Pty Ltd	Tower One, Level 46, 100 Barangaroo Ave, Sydney NSW 2000 Compliance@meetdandy.com	ABN: 33693955761	TGA (Therapeutic Goods Administration)
EU — English	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	Competent Authority / EUDAMED
EU — España	Dandy Labs Europa	3, boulevard de Sebastopol, 75001 París, Francia Compliance@meetdandy.com	Reg: 989857065	AEMPS / Autoridad Competente
EU — France	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, France Compliance@meetdandy.com	Reg: 989857065	ANSM / Autorité Compétente
EU — Deutschland	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Paris, Frankreich Compliance@meetdandy.com	Reg: 989857065	BfArM / Zuständige Behörde
EU — Italia	Dandy Labs Europe	3, boulevard de Sebastopol, 75001 Parigi, Francia Compliance@meetdandy.com	Reg: 989857065	Min. della Salute / ISS
United Kingdom (UK)	DANDY LABS GB, LTD	5 New Street Square, London EC4A 3TW Compliance@meetdandy.com	Co. No.: 16873608	MHRA

NOTE

Contact details are subject to change. Always verify current information on the official product labelling or the manufacturer's website before reporting an incident. / Les coordonnées sont susceptibles de changer. / Angaben können sich ändern. / I recapiti sono soggetti a modifiche.

**Manufacturer / Hersteller /
Fabbricante / Fabricant /
Fabricante**

Zima International, Inc. Also DBA Dandy
1320 N. 300 W., Lehi, UT 84043, United States
compliance@meetdandy.com

This is a representation of an electronic record that was signed electronically and this page is the manifestation of the electronic signatures.

Signatory Table

Action Name	User Name	Title	Signature Date
Approve	Kasha Lim	Supplier Quality Engineer	12-Jun-2026 01:03

* Datetime fields are displayed according to the system time zone: (GMT-04:00) Eastern Daylight Time (America/New_York)